

Table of Contents

Introduction	4
Labeling	5
Part I: Using your HiRes™ Auria™ Processor	7
Overview	7
HiRes™ Auria™ Processor Components	7
HiRes™ Auria™ Processor	7
Microphone	8
Program Switch	8
Volume Dial	8
Auria™ Headpiece with Attached Cable	9
Connecting the HiRes™ Auria™ Headpiece to the HiRes™ Auria™ Processor	9
Removing the HiRes™ Auria™ Headpiece from the HiRes™ Auria™ Processor	9
Auria™ Accent Color Covers and Headpiece Caps	9
Earhook Options	10
Standard Earhook	11
Auria™ T-Mic™	11
Auria™ Direct Connect	11
Auria™ FireFly™	13
Rechargeable Auria™ PowerCel™	14
Check Auria™ PowerCel™ Charge and Lock Status	14
Replacing the Auria™ PowerCel™	15
Auria™ PowerPak™	15
To Connect the Auria™ PowerPak™	16
To Disconnect the Auria™ PowerPak™	16
Replacing the Battery in the Auria™ PowerPak™	16
Using the Auria™ PowerPak™	16
Preparing the Auria™ PowerPak™ for use with the Plug or Huggie	16
Using the Telephone	17
Part II: Care and Maintenance of your HiRes™ Auria™ Processor	19
Overview	19
Cleaning	19
HiRes™ Auria™ Processor Protection	19
Avoid the following	19
Electro-Static Discharge	20
Dri-Aid Kit	21
Using the Dri-Aid Kit	21

Airport Security Metal Detectors	21
Rechargeable Auria™ PowerCels™	22
Auria™ PowerCel™ Wallet	22
Charger and Power Supply	22
Light Emitting Diode (LED) Indicators	23
Charging the Auria™ PowerCels™	23
Assembling the Charger	23
Inserting the Auria™ PowerCels™ into the Charger	24
Removing the Auria™ PowerCels™ from the Charger	25

Part III: Assistive Listening Devices 27

Overview	27
Personal FM Systems	27
Operating Instructions	27
Troubleshooting tips	27
Infrared Systems	28
Operating Instructions	28
Troubleshooting Tips	29
Hard-wired Systems	29
Operating Instructions	29
Troubleshooting Tips	29

Part IV: Troubleshooting your HiRes™ Auria™ Processor 31

No Sound Heard	31
Static Heard	31
Muffled or Distorted Sounds Heard	31
Auria™ Headpiece Cap or HiRes™ Auria™ Processor Gets Wet	32
Blinking Yellow LED on the Charger	32
No Green LED on the Charger	32
Problems with Adherence of the Auria™ Headpiece Cap to the Head	33
Maintaining Stability Behind the Ear	33

Part V: Contact Us 35

Table of Figures

FIGURE 1: HiRes™ Auria™ Processor with Components.....	7
FIGURE 2 & 3: Program Switch and Volume Dial.....	8
FIGURE 4: HiRes™ Auria™ Processor with Connection to Attached Headpiece and Cable.....	9
FIGURE 5: Removal of Auria™ Headpiece Cap.....	9
FIGURE 6: Removal of HiRes™ Auria™ Processor Accent Color Cover.....	10
FIGURE 7: Auria™ T-Mic™.....	11
FIGURE 8: Auria™ Direct Connect Cable to HiRes™ Auria™ Processor.....	11
FIGURE 9: Auria™ Direct Connect with mono to stereo adapter for FM use.....	12
FIGURE 10: Connecting the cable to the Auria™ Direct Connect Earhook.....	13
FIGURE 11: Auria™ FireFly™.....	13
FIGURE 12: Removing Auria™ Earhooks.....	14
FIGURE 13: Replacing Auria™ Earhooks.....	14
FIGURE 14: Auria™ PowerPak™ with Huggie.....	15
FIGURE 15: Replacing the AA Battery in the Auria™ PowerPak™.....	16
FIGURE 16: HiRes™ Auria™ Processor with Huggie.....	17
FIGURE 17: Charging Bay on Flat Surface.....	23
FIGURE 18: Power Supply Adapters.....	23
FIGURE 19: Connecting the Adapters to the Power Supply.....	24
FIGURE 20: Connecting the Power Supply to the Charger.....	24
FIGURE 21: Inserting Auria™ PowerCel™ into Charger.....	24
FIGURE 22: Removing the Auria™ PowerCel™ from the Charger.....	25

Introduction

The User Guide provides information on the use and maintenance of the HiRes™ Auria™ Processor.

HiRes™ Auria™ Processor Kit Contents








The following items may or may not be included in your HiRes™ Auria™ Processor Kit. Please refer to the contents list for the actual contents of your kit:

HiRes™ Auria™ Processor	CI-5230
Auria™ Headpiece	CI-5304
Auria™ T-Mic™	CI-5830
Auria™ Direct Connect	CI-5840
Auria™ Direct Connect Cable	CI-5841
Auria™ FireFly™	CI-5720
Accent Color Covers — Auria™ PowerCel™ Slim	CI-7120
Accent Color Covers — Auria™ PowerCel™ Plus	CI-7121
Accent Color Covers — Auria™ PowerPak™	CI-7122
Auria™ Accent Color Headpiece Cap	CI-7123
Auria™ PowerPak™	CI-7400
Auria™ Huggie	CI-7450
Auria™ PowerCel™ — Slim	CI-5520
Auria™ PowerCel™ — Plus	CI-5540
Auria™ PowerCel™ Charger Bay	CI-5600
Auria™ PowerCel™ Charger Power Supply	CI-5610
Auria™ PowerCel™ Charger Power Supply Adapters	CI-5630
Auria™ PowerCel™ Wallet	CI-7115
Auria™ User Video	7095544-001

Specifications

Labeling

The symbols below are used on the labeling for the product and for transportation, and their meanings are as follows:

CAUTION: Federal law restricts this device to sale, distribution and use by or on the order of a physician.	
 European Community Mark of Conformity Authorized to affix the CE Mark in 2003	
REF Model Number	SN Serial Number
 See Instructions for Use	 Store at temperatures between 0° C and +50° C
 Fragile	 XX-XXXX Date of Manufacture
 Type of Protection: BF	 Do not get wet


EN60601-1 Classification Information:
 Ordinary Construction
 Continuous Operation

Part I: Using your HiRes™ Auria™ Processor

Overview

The HiRes™ Auria™ Processor is a small lightweight, behind the ear sound processor designed to work with the most advanced technology in sound processing. The HiRes™ Auria™ Processor is designed with easy to use controls to suit a variety of individual needs.

FIGURE 1: HiRes™ Auria™ Processor with Components

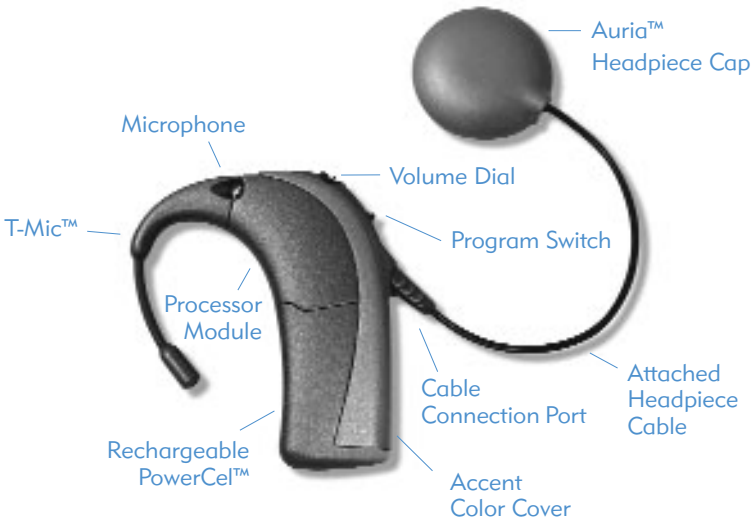

HiRes™ Auria™ Processor Components

HiRes™ Auria™ Processor:

The HiRes™ Auria™ Processor module houses the operating electronics. The headpiece and cable attaches to the cable connection port located beneath the program switch on the back of your HiRes™ Auria™ Processor. To turn the HiRes™ Auria™ Processor on attach the Auria™ PowerCel™ or Auria™ PowerPak™. To turn it off remove the Auria™ PowerCel™ or Auria™ PowerPak™.

Microphone:

The HiRes™ Auria™ Processor utilizes a forward facing omni-directional (accepts sound from all directions) microphone located at the top of the processor to capture sound input.

Program Switch:


The HiRes™ Auria™ Processor can store up to three programs. The program switch is located below the volume dial on your processor and has three positions to represent each stored program:

Program 1: switch all the way down

Program 2: switch in the middle

Program 3: switch all the way up

FIGURES 2 & 3: Program Switch and Volume Dial

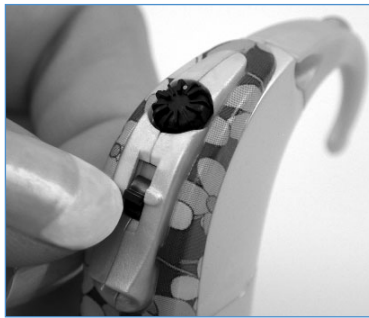

Volume Dial Operation:

The volume dial can be used to control the loudness level of a program. To adjust the volume:

- Turn dial clockwise using the side of your index finger to increase volume.
- Turn dial counterclockwise using the side of your index finger to decrease volume.




NOTE: Usually your audiologist will set the 12 o'clock position on your volume dial as your most comfortable listening level. The volume position is indicated by the white dot located on the dial.

CAUTION: To avoid overly loud sounds please remember to turn the volume dial down prior to placing the HiRes™ Auria™ Headpiece over the implant.

Auria™ Headpiece with Attached Cable

The headpiece contains the transmitter for sending sound information to the implant. The headpiece is held securely in place over the implant by magnetic attraction. It is designed with smooth rounded surfaces of lightweight materials to provide comfort for users of the HiRes™ Auria™ Processor. The headpiece cable connects the HiRes™ Auria™ Processor just below the program switch.

FIGURE 4: HiRes™ Auria™ Processor with Connection to Attached Headpiece and Cable

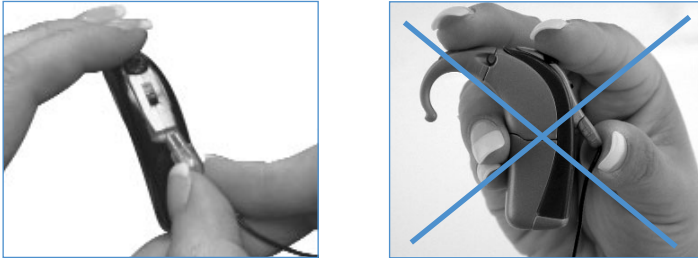

Connecting the HiRes™ Auria™ Headpiece to the HiRes™ Auria™ Processor:

- Align the raised area on the end of the cable's strain relief with the groove on the cable connector on the body of the processor.
- Slide in place until secure.

Removing the HiRes™ Auria™ Headpiece from the HiRes™ Auria™ Processor:

Hold the cable's strain relief (the plastic plug, not the wire) and gently pull it away.

Auria™ Accent Color Covers and Headpiece Caps

The HiRes™ Auria™ Processor is designed to allow for changes in accent colors both for the processor and the headpiece. Three types of accent color covers are available for the processor based on the power system that you have chosen (Auria™ PowerCel™ Slim, Auria™ PowerCel™ Plus or the Auria™ PowerPak™). The color caps adhere to the headpiece by magnetic attraction.

FIGURE 5: Removal of Auria™ Headpiece Cap



Removing the Auria™ Headpiece Cap:

- Place index finger underneath Auria™ Headpiece Cap
- Hold cable gently at the base of the Auria™ Headpiece with one thumb
- Slide other thumb under the raised portion of the color cap
- Gently pull up

Replacing the Auria™ Headpiece Cap:

- Align color cap with Auria™ Headpiece
- Gently place cap onto the headpiece until it magnetically attracts

Removing the Auria™ Accent Color Cover:


- Unplug cable
- Slide finger between the color cover at the bottom of the processor
- Gently pull up and slide forward

Replacing the Auria™ Accent Color Cover:

- Align the color cover at the top of the processor
- Gently push on

WARNING: Small Parts Hazard: Parents and caregivers should be counseled that the external implant system contains small parts. The Auria™ Accent Color Covers, Auria™ Headpiece Caps, the metal contact on the inside of the headpiece cap, and earhooks may cause choking if swallowed.

FIGURE 6: Removal of HiRes™ Auria™ Processor Accent Color Cover

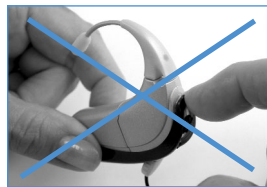

Earhook Options

NOTE: Use of the various earhook accessories may require different audio-mixing ratios. Please consult your audiologist for evaluation of the most satisfactory mixing ratio and subsequent programming adjustment for the earhook in use.

Four earhook options are available to support your HiRes™ Auria™ Processor.

1. Auria™ Standard Earhook

When using the Auria™ Standard Earhook, the microphone located at the top of the processor is the input source to the processor. An earmold may be used with the standard earhook to provide added security of the processor on the ear.

2. Auria™ T-Mic™

The Auria™ T-Mic™ is an integrated earhook and microphone combination. The microphone is omni-directional (accepts sound from all directions) and is designed to fit in the open bowl portion of your ear. The Auria™ T-Mic™ is appropriate for everyday use and may provide added benefit in difficult listening situations, such as in noisy environments or on the phone. For the best results, some listeners may require an auxiliary only program while using the Auria™ T-Mic™.

NOTE: When fitting the Auria™ T-Mic™ over the ear, bend by placing finger near the microphone and pushing gently. Avoid sharp angles (90° or greater) as it can shorten the lifespan of the Auria™ T-Mic™.

FIGURE 7: Auria™ T-Mic™



2. Auria™ Direct Connect

The Auria™ Direct Connect Earhook allows you to connect to auxiliary audio input devices such as a CD player or FM systems. It requires the use of the Auria™ Direct Connect Cable and audio interface cable.

NOTE: Consult with your audiologist for the appropriate audio mixing ratio settings.

FIGURE 8: Auria™ Direct Connect Cable to HiRes™ Auria™ Processor



Auria™ Direct Connect Cable

This cable comes in 3 different lengths and is designed to connect to a variety of assistive listening devices such as FM Systems, Infrared Systems, and CD Players. These audio sources typically require a custom-made patch cable available either from the manufacturer of these devices or from an electronics store. The cable has a 3.5mm stereo audio jack at one end and a custom miniature connector at the other end that plugs into a connector on the top of the earhook. With or without the Direct Connect Cable attached, the quick disconnect earhook mechanically functions as a standard earhook, therefore you can attach an earmold with standard tubing to the end of this earhook. When programmed appropriately the Auria™ Direct Connect earhook can function as a standard earhook.

NOTE: When using a mono source such as a FM System you must use a mono to stereo adapter (included in the Auria™ Direct Connect Cable package).

FIGURE 9: Auria™ Direct Connect with mono to stereo adapter for FM use



To Connect Auria™ Direct Connect Cable:

- Remove HiRes™ Auria™ Processor from ear
- Hold the HiRes™ Auria™ Processor by the earhook
- Gently push the Auria™ Direct Connect Cable into the port at the top of the earhook.

Below is an alternate method for attaching the Auria™ Direct Connect Cable to the Auria™ Direct Connect Earhook:

- Rotate the connector 180 degrees
- Approach the Auria™ Direct Connect at an angle
- Snap into place
- Rotate the connector 180 degrees to align

To Disconnect the Auria™ Direct Connect Cable:

- Remove the HiRes™ Auria™ Processor from ear
- Hold the HiRes™ Auria™ Processor by the base of the earhook
- Gently pull the Auria™ Direct Connect Cable from the port at the top of the earhook.

FIGURE 10: Connecting the cable to the Auria™ Direct Connect Earhook



3. Auria™ FireFly™

The Auria™ FireFly™ is an illuminated earhook that provides visual feedback of the processor functions and program settings for parents and educators.

- The FireFly™ will blink the number of times that correspond to the program in use (i.e. If program two is in use the FireFly™ will blink two times).
- The FireFly™ will light up steadily if the HiRes™ Auria™ Processor is successfully communicating with the implant.
- The FireFly™ will blink on and off rhythmically if the HiRes™ Auria™ Processor is not communicating with the implant.
- The FireFly™ will not blink at all if the Auria™ PowerCel™ or Auria™ PowerPak™ power supply is depleted or disconnected.

NOTE: Consult with your audiologist for the appropriate audio mixing ratio settings.

FIGURE 11: Auria™ FireFly™



To remove earhooks:

- Turn the earhook accessories slightly more than a 1/4 turn in either direction until it pops off.

FIGURE 12: Removing Auria™ Earhooks



To replace earhooks:

- To replace the earhook accessories
- Align the earhook so it is flush with your HiRes™ Auria™ Processor.
- Check to make sure the orange o-ring is in place at the base of the HiRes™ Auria™ Processor. Hold the earhook at the base.
- Push on until it lightly snaps in place.
- Gently pull to confirm that it is locked in place.

NOTE: If the orange o-ring is missing from your HiRes™ Auria™ Processor, you will need to exchange your processor. The earhook will not snap into place without the o-ring.

FIGURE 13: Replacing Auria™ Earhooks



Rechargeable Auria™ PowerCel™

A custom lithium-ion rechargeable Auria™ PowerCel™ powers your processor. The rechargeable Auria™ PowerCel™ is available in two sizes (Slim and Plus) with different operating and charging times.

Check Auria™ PowerCel™ Charge and Lock Status

In order to check the Auria™ PowerCel™ charge and lock status in children, observe the status of the Auria™ FireFly™ (refer to Auria™ FireFly™ description under Earhook Options).

Replacing the Auria™ PowerCel™:

- Slide the Auria™ PowerCel™ forward while gently holding the processor and color cover.
- Guide the Auria™ PowerCel™ onto the tracks on the HiRes™ Auria™ Processor.
- Slide the Auria™ PowerCel™ onto the processor until it stops and the Auria™ PowerCel™ and the processor are aligned.

NOTE: Do not force the Auria™ PowerCel™ onto the processor. The Auria™ PowerCel™ is designed to be inserted in only one direction. Forcing the PowerCel™ may jam or damage the tracks.

NOTE: To prevent damage to the Auria™ T-Mic™ be sure to turn it gently out of the way prior to removal of the Auria™ PowerCel™.

NOTE TO EUROPEAN CUSTOMERS: For proper disposal of rechargeable Auria™ PowerCel™ modules, please return depleted Auria™ PowerCel™ modules to the local Advanced Bionics Corporation representative or Advanced Bionics SARL.

Auria™ PowerPak™

The Auria™ PowerPak™ offers an alternative to the rechargeable Auria™ PowerCel™. The Auria™ PowerPak™ can be worn on clothing or on a belt or in a pocket. The Auria™ PowerPak™ uses one non-rechargeable AA battery (preferably Duracell or Energizer). You may opt to use a rechargeable Nickel Metal Hydride for best performance. There will be a slight increase in overall weight (4gr) with the use of this rechargeable technology. Inserting a battery into the Auria™ PowerPak™ activates the HiRes™ Auria™ Processor. The battery should be removed when the HiRes™ Auria™ Processor is not in use to prevent further depletion of the battery.

WARNING: Follow the recommendations of the battery manufacturer never attempt to charge, non-rechargeable batteries in the charger.

FIGURE 14: Auria™ PowerPak™ with Huggie



To connect the Auria™ PowerPak™:

- Slide the Auria™ PowerCel™ off the processor
- Remove color cover
- Slide the Auria™ PowerPak™ onto the tracks of the processor and push latch up to lock
- Place short color cover on processor

To disconnect Auria™ PowerPak™:

- Remove the Auria™ Headpiece and cable
- Remove color cover
- Pull latch down to unlock by placing your fingers on either side of the latch or your fingernail in the groove at the bottom of the latch.
- Slide Auria™ PowerPak™ off processor

Replacing the Battery in the Auria™ PowerPak™:

- Unscrew the Auria™ PowerPak™
- Remove the battery
- Replace battery making sure the negative (flat) end of the battery is positioned toward the spring in the Auria™ PowerPak™. The Auria™ PowerPak™ is marked with a (+) sign to indicate proper direction of the battery.

FIGURE 15: Replacing the AA Battery in the Auria™ PowerPak™



Using the Auria™ PowerPak™

The Auria™ Huggie can be used to help stabilize the HiRes™ Auria™ Processor on the ear while using the Auria™ PowerPak™.

Preparing your Auria™ PowerPak™ for use with the Plug or Huggie

To attach the Auria™ PowerPak™ Plug or Huggie:

- Gently push the Auria™ PowerPak™ Plug or Huggie into open slot at the bottom of the Auria™ PowerPak™.
- Place screw in top of Plug or Huggie
- Using #2 or 1/8" flat blade screwdriver gently tighten the screw on top of the Auria™ PowerPak™ adapter.
- Follow directions for attaching Auria™ PowerPak™ to HiRes™ Auria™ Processor.

FIGURE 16: HiRes™ Auria™ Processor with Huggie



To remove the Auria™ PowerPak™ Plug:

- Loosen the screw but do not remove
- Gently push on screw until plug is loosened and easy to grasp
- Remove screw
- Gently pull plug out

To remove the Auria™ PowerPak™ Huggie:

- Loosen the screw
- Gently slide Huggie out

NOTE: When the HiRes™ Auria™ Processor with Auria™ PowerPak™ is not in use, be sure to remove the battery from the Auria™ PowerPak™ otherwise, the battery will continue to drain.

Using the Telephone

Your audiologist will indicate when you should begin to use the telephone with your HiRes™ Auria™ Processor. You can use the telephone by placing the receiver directly over the microphone located in the earhook of your HiRes™ Auria™ Processor or directly over your ear when wearing the Auria™ T-Mic™. You may find the Auria™ T-Mic™ beneficial for you in particularly noisy situations.

Practice with several phones, some with adjustable volume, to find what works best for you. Initially, you may want to practice using the telephone with your audiologist or a familiar person. Be patient. Telephone communication with the implant often improves over time as you gain experience using the device.

Using or being in the vicinity to someone who is using certain digital cellular phones may cause interference with the cell phone's reception or problems with the quality of sound from your device. If such interference occurs, you can remove your HiRes™ Auria™ Processor or move away from the source. Before purchasing a digital cellular phone, you should evaluate whether or not interference is present. GSM cellular or cordless phones create the most audio interferences and may require a remote

adapter that can be obtained from the cell phone company. No such interference has been noted with cellular phones that use older analog technology or CDMA digital service.

Part II: Care and Maintenance of your HiRes™ Auria™ Processor

Overview

Although your HiRes™ Auria™ Processor has been designed and built to withstand daily wear and tear, care must be taken to protect the external components of the device. For a detailed discussion of warnings and precautions please refer to the Package Insert located at the back of this guide. It is a good idea to carry the User Identification Card with you at all times.

Cleaning

If necessary, the HiRes™ Auria™ Processor and Auria™ Headpiece can be cleaned with a slightly dampened cloth or tissue. Take care that water does not drip into any connectors or the microphone. Water or other fluids should never enter the HiRes™ Auria™ Processor or the Auria™ Headpiece.

CAUTION: Immersion in water will damage the HiRes™ Auria™ Processor and Auria™ Headpiece electronics.

To prevent intermittent operation of the processor, the Auria™ PowerCel™ contacts on the rechargeable Auria™ PowerCel™ and HiRes™ Auria™ Processor should be kept free from dirt, dust, perspiration or moisture. Clean the contacts with a dry cotton swab at least once a month and immediately after exposure to moisture or perspiration.

HiRes™ Auria™ Processor Protection

Your HiRes™ Auria™ Processor contains advanced electronics that can be damaged. Care should always be taken when using or handling. If you drop your processor, check that it is functioning properly. If you suspect that your processor has been damaged, contact your cochlear implant center.

Avoid the following:

- Dropping the HiRes™ Auria™ Processor.
- Wearing the HiRes™ Auria™ Processor while bathing, showering or swimming.
- Exposing the HiRes™ Auria™ Processor to extreme temperatures (below 32° F/0° C or above 115° F/ 45° C e.g., a closed car on a hot day or near a heater or radiator).

Electro-Static Discharge

Your HiRes™ Auria™ Processor and its attachments are designed to withstand most static events without damage or interruption to your programs. However, there is a chance that high levels of static electricity will generate extreme electrostatic discharge events that will damage the headpiece or sound processor of cochlear implant systems. Therefore, care should be taken to avoid exposing the headpiece, cables or sound processor to situations in which high levels of static electricity are created (static electricity will not damage the implant which is isolated under the skin).

Electro-Static Discharge (ESD) is present when you feel a spark jump from your body. ESD events are more likely to occur in very dry or cold environments. ESD events are less common in humid areas.

High levels of electro-static charge are known to build up under the following circumstances:

- Walking on carpets
- Sliding on plastic slides
- Exiting an automobile
- Pulling sweaters on and off
- Touching TV or computer screens
- Removing bedding materials

You should take the following basic precautions in order to reduce ESD to your processor system:

- Static electric potential can be safely reduced by touching any person or object with your fingers prior to that person or object contacting a headset, cables or sound processor. Hand contact will safely equalize electric charge and prevent sparks from jumping to the processor system. When approached by someone (for example on carpet), touch him or her first with your hand prior to them touching your external components. Parents are advised to touch their child before they touch a headset or sound processor that their child is wearing. Also, before placing any part of a processor system on a table or other surface, you should first touch the surface.
- Avoid contact between your processor/cables/headpiece and metal surfaces before you touch the metal surface with your hand. This is particularly important on the playground where high levels of static electricity can be easily generated and extreme electrostatic discharge events can occur between a headset or sound processor and metal that supports the playground equipment. A plastic tubular slide, for example, will quickly charge a child who then often immediately runs to a metal support structure such as a ladder. If at all possible, teach children who wear cochlear systems to “touch with hands first.”
- Carefully exit motor vehicles, particularly in dry, cold weather. Try to avoid contact between processor system components and any vehicle metal as you exit.

- When possible, remove your external equipment prior to removing sweaters.
- The screens of televisions and computer monitors are highly charged and contact is a problem for two reasons. First, the charge on screens can cause sparks. Second, brushing or touching a screen can transfer charge that will then discharge when you come in contact with another object or person.
- Use fabric softener when washing clothes and bedding, this will reduce the likelihood of charge generation. If you strip a bed while wearing a processor system, please remember to use your hands to safely reduce any charge buildup you may have acquired.

REMEMBER: “Touch with hands first!”

Dri-Aid Kit

A Dri-Aid Kit is included in your HiRes™ Auria™ Processor kit. Regularly place the HiRes™ Auria™ Processor and headpiece into the Dri-Aid Kit to help prevent moisture build-up. The Dri-Aid Kit includes the plastic bag and crystal container.

Using the Dri-Aid Kit:

Remove the labels on both sides of the crystal container to expose the crystals.

Observe the color of the crystals through the openings.

If they are blue: The crystals are ready to be used.

NOTE: If the crystals have not been used for a while, they may appear dark in color; they are fine and can be used.

If they are Pink: You will need to regenerate the crystals.

Regenerating the crystals: Place the crystal container in a pre-heated oven 350° F/176° C for 30 – 45 minutes until the crystals have turned blue. The crystals can be regenerated indefinitely. However if the crystals remain pink after heating, you must replace the Dri-Aid-Kit.

Remove the battery and place the crystal container, Auria™ Headpiece and the HiRes™ Auria™ Processor in the plastic bag.

Leave in the plastic bag overnight. The crystals turn pink as moisture is absorbed.

Airport Security Metal Detectors

Metal detectors, x-ray machines, and security scanners will not damage the implant or processor. However, the implant system may activate the detector alarm, or users

may hear a distorted sound when passing through a security door or during a hand-held wand scan. Turning the processor volume down will ensure that any sounds heard are not too loud or uncomfortable.

Rechargeable Auria™ PowerCels™

When you are not using your HiRes™ Auria™ Processor, you should remove the rechargeable Auria™ PowerCel™.

Auria™ PowerCel™ capacity decreases with age and usage. This is normal for all rechargeable Auria™ PowerCels™ and should not be considered a defect.

If the rechargeable Auria™ PowerCel™ is dropped, look at it for evidence of damage or cracking. If any evidence of damage is seen, the Auria™ PowerCel™ should be disposed of immediately.

WARNING: The Auria™ PowerCel™ used in this device may present risk of fire or chemical burn if mistreated. Do not disassemble, heat above 100° C / 212° F or incinerate. Dispose of used Auria™ PowerCels™ appropriately. Never put PowerCels™ in mouth. If swallowed, contact your physician or local poison control center immediately.

WARNING: To prevent injury or burns due to a short circuit, do not allow metal objects, such as keys or coins, to contact the battery terminals. Auria™ PowerCel™ Wallets are provided to carry and protect the batteries.

Auria™ PowerCel™ Wallet

An Auria™ PowerCel™ Wallet is provided with each HiRes™ Auria™ Processor. The wallet allows you to carry an additional Auria™ PowerCel™ safely and conveniently in your pocket or in your purse.

Charger and Power Supply

The charger provided with the HiRes™ Auria™ Processor is designed to recharge up to four Auria™ PowerCels™ simultaneously. It takes a little over 4 hours to fully recharge an Auria™ PowerCel™ depending upon size of Auria™ PowerCel™ used (Slim vs. Plus).

NOTE: Auria™ PowerCels™ do not need to be fully depleted before recharging.

CAUTION: Only use the charger for charging Advanced Bionics Auria™ PowerCels™. **DO NOT** use it to charge other batteries. Do not try to charge Auria™ PowerCels™ using a charger other than the one supplied by Advanced Bionics.

Light Emitting Diode (LED) Indicators:

A green light illuminates when the charger is connected to a power source and indicates power flowing to the unit. Along the side of the charger are 4 light indicators, one for each battery that indicates the charge status. The light is yellow when the Auria™ PowerCel™ charging cycle is in progress. The light turns off when the Auria™ PowerCels™ are fully charged.

Charging the Auria™ PowerCels™:

The following section describes assembling the charger and inserting the rechargeable Auria™ PowerCels™ in the charger.

Assembling the Charger:

Place the charger on a flat surface:

FIGURE 17: Charging Bay on Flat Surface




Select the power plug appropriate for your location.

FIGURE 18: Power Supply Adapters



Connect the power plug to the power supply.

FIGURE 19: Connecting the Adapters to the Power Supply

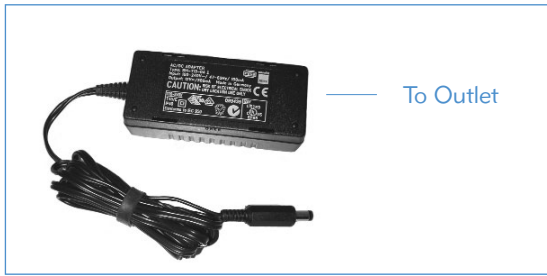

Connect the cable from the power supply to the charger.

FIGURE 20: Connecting the Power Supply to the Charger



Insert the power plug into the wall outlet. A green light indicates proper connection.

Inserting the Auria™ PowerCels™ into the Charger:

Locate the slide tracks on the charger.

FIGURE 21: Inserting Auria™ PowerCel™ into the Charger



Position the Auria™ PowerCel™ so that the contact is facing the vertical wall where the pins reside.

Slide the Auria™ PowerCel™ onto the charger until it engages.

You can insert up to four Auria™ PowerCels™ in the charger at one time. The charger is designed so the Auria™ PowerCels™ can only be inserted in one direction.

Auria™ PowerCels™ will start charging automatically. The yellow light indicator will illuminate to signal that charging is in progress.

When the Auria™ PowerCel™ is fully charged, the light for that particular Auria™ PowerCel™ will turn off.

NOTE: If you place a partially charged Auria™ PowerCel™ in the charger, the Auria™ PowerCel™ may be fully charged in a shorter period of time. You can remove the Auria™ PowerCel™ at any time. The charger and Auria™ PowerCels™ will not be damaged if the Auria™ PowerCels™ remain in the charger longer than the required charging time.

It is a good idea to rotate the Auria™ PowerCels™ in daily use in order to maximize Auria™ PowerCel™ life.

You must recharge your Auria™ PowerCels™ at least once every 3 months even if you are not using them.

Removing the Auria™ PowerCels™ from the Charger:

Gently press down the tab on the top of the charger above the Auria™ PowerCel™. Slide the Auria™ PowerCel™ up and out of the slide tracks of the charger.

FIGURE 22: Removing the Auria™ PowerCel™ from the Charger



The rechargeable Auria™ PowerCels™ and charger contacts should be kept free from dirt and dust. Dirty contacts can result in charger malfunction. Clean the contacts at least once a month using a dry cotton swab.

If the charger or power supply is dropped, inspect it for evidence of damage or cracking. If any evidence of damage is seen, the charger or power supply should be replaced. Prior to use inspect the charger cable to ensure that it is not frayed or damaged and that the plug is not broken. If your power cord appears to be damaged, it should be replaced.

Part III – Assistive Listening Devices

Overview

Assistive listening devices such as FM systems and Infrared systems can help to improve speech understanding, particularly in difficult listening situations. These devices reduce the effect of distance between the sound source and the listener.

Personal FM Systems

The HiRes™ Auria™ Processor can be used with a personal FM system by connecting the FM system to the Auria™ Direct Connect Cable. While some FM systems require a custom patch cable, other FM receivers may connect to the Auria™ Direct Connect Cable without use of an additional patch cable.

It is important to obtain the appropriate patch cable for the processor type and FM type. The use of a patch cable other than those specified by either Advanced Bionics or the FM manufacturer may cause degradation of the audio signal sent to the processor.

FM systems use a radio frequency link between the transmitter and the receiver in order to transmit information. With the HiRes™ Auria™ Processor it is possible to adjust the level of input from the FM receiver and from the HiRes™ Auria™ Processor microphone. Your implant audiologist can select a variety of "audio-mixing" options depending on the individual needs on a program-by-program basis. Audio-mixing capabilities provide the option of how much sound (in percentage) is received from different microphone sources. The default audio-mixing ratio for the HiRes™ Auria™ Processor is 50/50. If applicable, your audiologist will set the FM system output, gain and tone controls per the FM manufacturer's recommendations prior to FM use.

Operating Instructions

If applicable to the FM system chosen:

- Set the FM volume control of the FM receiver to the minimum setting.
- Set the volume dial of the HiRes™ Auria™ Processor to the user's typical settings.
- In sequence turn on the FM transmitter, then receiver and finally the HiRes™ Auria™ Processor in that order.
- Increase the FM gain/volume control to a comfortable listening level.

Troubleshooting Tips

- Ensure that the appropriate patch cable for the make and model of FM system is being used. Do not substitute cables with unknown properties without first consulting your implant-care professional, Advanced Bionics or the FM manufacturer.
- Proper care of the patch cable (i.e. avoid sharp bending of the cable) will maximize it's life.

Infrared Systems

Infrared systems use invisible light-waves to transmit signals such as speech or music. Infrared light is invisible as the wavelength of the signal is beyond the capability of our eyes to see. The transmitter/emitter uses the infrared light-waves to send a signal, such as speech or music, to an individual wearing a wireless infrared receiver. The infrared system may be a small personal system or a system comprised of large panels or light-emitting diodes (LED) and few individual receivers. In large group situations, these emitters are mounted high on a wall, and aimed at the perspective audience. Inside of an infrared receiver is an infrared sensitive receiving diode.

Some infrared receivers have an audio output socket that will allow the implant recipient to patch the signal directly into the audio-input socket of the sound processor. A specially designed patch cable can be obtained from a variety of sources including the manufacturer of the infrared systems.

Note that not all infrared receivers have body worn environmental microphones or have an extra audio socket for an auxiliary microphone. Thus your audiologist will help select an appropriate mixing ratio between the built-in HiRes™ Auria™ Processor microphone and the signal from the infrared receiver.

Operating Instructions

- Ensure that the appropriate patch cable has been obtained for the make and model of the infrared system and the HiRes™ Auria™ Processor.
- With the infrared system and processor in the off position attach the appropriate ends of the patch cable to their respective input/output sockets.
- Set the infrared volume dial of the receiver to the minimum setting.
- Set the volume dial of the HiRes™ Auria™ Processor to the user's typical use settings.
- In sequence, turn on the infrared system transmitter, receiver and processor in that order.
- Increase the infrared system volume dial to a comfortable listening level or to a specifically recommended level by the infrared system manufacturer.

Troubleshooting Tips

- Ensure that the appropriate patch cable for the make and model of infrared system is being used. Do not substitute cables with unknown properties without first consulting Advanced Bionics or the infrared systems manufacturer.
- When the infrared systems manufacturer and/or Advanced Bionics have made specific setting recommendations, they have been made under specific conditions.
- Certain environmental conditions, such as the presence of large objects blocking the pathway of infrared systems light from the infrared systems transmitter to the infrared systems receivers can cause interference. Consider moving to another location in the environment.
- Proper care for the patch cable (i.e. avoid sharp bending of the cable) will maximize its life.

Hard-wired Systems

A hard-wired system can be described as any system that provides a direct physical connection between the sound source and the listener. Examples of this type of system include the use of the Auxiliary Microphone available through Advanced Bionics, pass around microphones with long extension cables, and patch cables for use with battery-powered appliances such as Walkman and CD players. While this type of assistive technology is typically the least expensive and the easiest to implement it has its limitations. There is the “safety factor” of having long cables to trip over and the distance limitation before the audio signal will become degraded due to electrical losses from the long cable lengths. Further, long cables tend to act as an antenna, and if not properly designed and can pick up unwanted signals.

CAUTION: One must not plug AC powered devices (e.g. TV, Stereo, Computer) into the HiRes™ Auria™ Processor without the use of an approved patch cable that has electrical surge protection built into its design. Ignoring this caution could lead to damage to the sound processor and/or to the recipient with unpleasant sensations or sounds.

Operating Instructions

- As with any patch cable one must determine if the hard-wired system has a plug that will fit into the audio-socket of the processor. The HiRes™ Auria™ Processor uses a standard 3.5mm socket. The audio socket of the processor is designed to accept a stereo input. While you may not be able to find a patch cable for all audio devices such as CD players, tape recorders, etc. At an audio appliance outlet, there are a number of third-party vendors that can provide an appropriately made cable for your needs. Please contact Advanced Bionics for a list of these vendors.

- Ensure that the appropriate patch cable has been obtained for the make and model of audio-appliance (i.e. CD player, Walkman) and HiRes™ Auria™ Processor.
- With the audio-appliance in the off position and the processor turned down, attach the appropriate ends of the patch cable to their respective input/output sockets. The ends of the cable should be appropriately marked, as the attenuated end of any cable should be closest to the processor.
- If applicable to the audio-appliance set any volume and tone controls per the manufacturer's recommendations.
- Set the volume dial of the audio-appliance to the minimum setting.
- Set the volume dial of the HiRes™ Auria™ Processor to the typical use settings.
- Turn on the audio-appliance and turn up the processor in that order.
- Increase the volume dial of the audio-appliance to a comfortable listening level or to a specifically recommended level by the manufacturer.

Troubleshooting Tips

Ensure that the appropriate patch cable for the make and model of the audio-appliance is being used. Do not substitute cables with unknown properties without first consulting the implant-care professional or Advanced Bionics.

- When specific setting recommendations have been made by the audio-appliance manufacturer and/or Advanced Bionics, they have been made under specific conditions.
- Long cables tend to become tangled around objects. Take care when draping long patch cables across busy areas at home and at work.
- Some audio-appliances were designed in such a fashion that plugging a cable into the audio-output/earphone port will disconnect the main speaker. Thus, others may not be able to hear the stereo while you are connected using your specially manufactured patch cable.
- Proper care of the patch cable (i.e. avoid sharp bending of the cable) will maximize its life.

Part IV: Troubleshooting Your HiRes™ Auria™ Processor

The following is a description of how to troubleshoot your HiRes™ Auria™ Processor. If a problem persists after trying the suggestions below, contact your implant center, service center or Advanced Bionics for support.

No Sound Heard

Ensure that the cable is inserted into the HiRes™ Auria™ Processor and the Auria™ Headpiece is properly positioned over the implant site. Ensure that you are using a charged Auria™ PowerCel™.

If the Auria™ PowerCel™ is charged:

Try the Auria™ T-Mic™ using a program that allows auxiliary input.

If this resolves the problem, place the HiRes™ Auria™ Processor in the Dri-Aid Kit overnight. If there is still no sound heard without the Auria™ T-Mic™, there may be an issue with the internal microphone. You should contact your implant center audiologist.

Static Heard

Clean the battery contacts as well as the contacts on the HiRes™ Auria™ Processor. Try the Auria™ T-Mic™ using a program that allows auxiliary input. If this fixes the problem, place the processor in the Dri-Aid Kit overnight. If there is still static heard without the Auria™ T-Mic™, there may be an issue with the internal microphone. You should contact your implant center audiologist.

Muffled or Distorted Sounds Heard

Remove any material (hat, headbands, etc), which may be covering the HiRes™ Auria™ Processor microphone.

Try the Auria™ T-Mic™ using a program that allows auxiliary input.

If this fixes the problem, place the processor in the Dri-Aid Kit overnight. If there is still muffled or distorted sounds without the Auria™ T-Mic™, there may be an issue with the internal microphone. You should contact your implant center audiologist. Also, try switching to another program to see if this resolves the problem.

Auria™ Headpiece or HiRes™ Auria™ Processor Gets Wet

Remove the Auria™ PowerCel™ immediately. Drain as much excess water as possible from the processor or headpiece.

Place the processor and/or Auria™ Headpiece in the Dri-Aid Kit immediately.

Contact your cochlear implant center or Advanced Bionics for further instructions.

NOTE: If your system appears to be working but you experience deterioration in the volume or the quality and clarity of sound, contact your implant center audiologist as your processor may need to be reprogrammed.

IMPORTANT: If exposure to fluids is suspected, attempt to dry processor in a warm room for 24 hours. **DO NOT** try to speed up the drying process with the use of a microwave oven, convection oven or hairdryer; as these may further damage the internal electronics.

WARNING: Your processor or headpiece should be serviced only at Advanced Bionics. Do not attempt to open or repair the processor. Do not continue to use the HiRes™ Auria™ Processor if any part of it is damaged. Unauthorized opening of the processor, or other equipment, will void the warranty and may compromise system performance.

Blinking Yellow Light-Emitting Diode (LED) on the Charger

A blinking yellow light indicates that the charger or Auria™ PowerCel™ is faulty. Try the Auria™ PowerCel™ in a different position. If the Auria™ PowerCel™ charges properly when moved to a new position, the charger may have a faulty position.

Contact your cochlear implant center or Advanced Bionics for further instructions.

No Green LED on the Charger

- Verify charger is connected to power.
- Replace power supply, adapters, and/or charger.

Problems with Adherence of the Auria™ Headpiece Cap to the Head

- Discuss the possibility of increasing magnet strength with your audiologist.

Maintaining Stability Behind the Ear

- Depending on the earhook option used, an earmold or commercially available huggie may provide additional security for the processor on the ear. Please consult your implant audiologist for their recommendation.

Part V: Contact Us

Advanced Bionics is committed to providing the highest quality products and service to our customers. We welcome your comments about the HiRes™ Auria™ Processor or your suggestions to improve the product. Please feel free to contact Advanced Bionics or discuss your suggestions with your implant professional.

Corporate Headquarters

Advanced Bionics Corporation
12740 San Fernando Road, Sylmar, California 91342, U.S.A
(800) 678-2575 within US and Canada
(661) 362-1400 • (661) 362-1500 Fax
(800) 678-3575 TDD
www.advancedbionics.com
Email: info@advancedbionics.com

European Headquarters

Advanced Bionics SARL
76, rue de Battenheim, 68170 Rixheim, France
+33 (0) 3-89-65-98-00 • +33 (0) 3-89-65-50-05 Fax
Email: europe@advancedbionics.com

Asia-Pacific

Advanced Bionics
25129 Rye Canyon Loop, Valencia, California 91355, U.S.A.
(661) 362-1400 • (661) 362-1500 Fax
Email: asiapacific@advancedbionics.com

Technical information will be provided upon request.

Advanced Bionics®, CLARION®, HiRes™, Auria™, T-Mic™, FireFly™, Auria™ PowerCel™ and Auria™ PowerPak™ and CIBTE™ are trademarks of Advanced Bionics Corporation in the United States of America and other countries.

This device is protected under one or more of the following U.S. Patents: 4,400,590, 4,405,831, 4,495,917, 4,686,765, 4,721,551, 4,819,647, 4,837,049, 4,931,795, 4,969,468, 4,990,845, 4,991,582, 5,443,493, 5,477,855, 5,513,793, 5,522,865, 5,531,774, 5,545,191, 5,569,307, 5,571,148, 5,584,869, 5,601,617, 5,603,726, 5,609,616, 5,626,629, 5,738,270, 5,776,172, 5,833,714, 5,876,425. Other U.S. and/or foreign patents are pending.